



# First Aid Products for burns

**AbsorBurn®**

# AbsorBurn®

## First Aid Hydrogel Products for burns

AbsorBurn is available in gel dressing (hydrogel coated foam) and in a tube of plain hydrogel.

- Effectively cools the burned area by decreasing the temperature with 6 - 7° C.
- Relieves pain.
- Is very easy to use.

## Directions of use



1. Open the package.



2. Apply the dressing on the burned area. Leave it on for 30-45 minutes.



3. Alternatively apply gel on the burned area, or if needed add gel to the dressing.

Caution! In severe burns always contact a physician.

## Product assortment

Product	Size	Number/pack
Gel dressing	5 x 5 cm	1
Gel dressing	10 x 10 cm	1
Gel dressing	20 x 20 cm	1
Gel tube	100 ml	1
Start kit *		1

\* Start kit contains of: dressing 10x10 cm 3 pcs, 20x20 cm 1 pc and gel tube 1 pc.

Ingredients: demineralised water, natriumhydroxide, propylene glycol, carbomer, preservatives, lavender oil and Tea Tree Oil.



Focus on health