


Sorbact[®] Foam

Microorganism binding foam dressing

NEW!

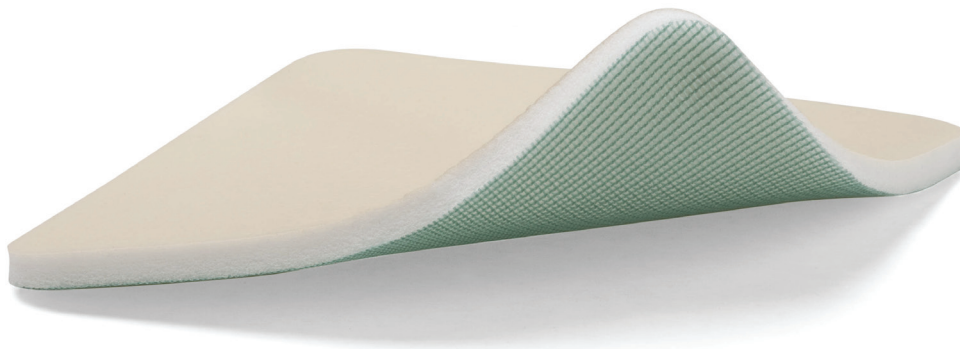

Green wound healing





Product Description

Foam dressing with Sorbact's bacterial and fungal binding property. The dressing is suitable for use in mild to moderate exuding wounds.

Sorbact Foam consists of acetate fabric coated with a unique specially designed microorganism binding substance, laminated on polyurethane foam with a semi permeable film backing. The dressing is conformable and pressure relieving.

Area of use

All types of exuding, contaminated to infected wounds:

- Pressure ulcers, leg ulcers, foot ulcers
- Acute traumatic wounds
- Surgical wounds

Benefits

- Bacterial control
- Fluid handling
- No risk of resistance development
- Conformable

User manual

Apply the dressing directly on the wound surface with the green side to the wound surface. Secure the dressing with suitable fixation.

Note that the green Sorbact fabric has to have direct contact with the wound surface.

The dressing can be cut in a desirable shape or size.

Application on areas such as heel or elbow, it is recommended to cut tabs in the corners.

Dressing changes, depending on exudate level, daily to twice weekly.

Product	Item no	Size	No/package
Sorbact Foam	98310	10 x 10	10
Sorbact Foam	98315	15 x 15	10
Sorbact Foam	98320	10 x 20	10

